

CD316SA

Gasting Handle

For Small Air-Lock

Weight limit 265 lbs

419 Curtis Rd, Boise, ID 83706

(208) 429-0026

www.coyotedesign.com

**Coyote
Design**



Advena Ltd
Pure Offices, Plato Close
Tachbrook Park
Warwick, CV34 6WE, UK

CE

External Prosthetic Components

Casting handle

For easier casting and fabrication, and to prevent incorrect lock placement, we recommend using the Casting Handle

If using a distal cushion, it is best if the cushion is cast into place. In some cases, memory foam cushions can be added after the fact

1. Install casting pin in liner of choice
2. Slide handle onto pin
3. From above and the side, make sure the liner/pin is in the desired location
4. Remove handle,
5. slide distal cushion over pin if being used, reinstall handle
6. Foam circles can be used to mark key areas (CD103SFC and CD103LFC)
7. Stockinette is put over the handle and liner
8. With first couple of wraps, go around handle to key into place, then continue to wrap rest of limb as usual
9. Continue to check location and angle of handle during casting to ensure the lock placement is as desired
10. Once cast is removed, install lock anchor, and fill cast
11. Handle has identical shape as lock for proper seating and location



Casting Handle and Pin for Air-Lock
CD316SA